
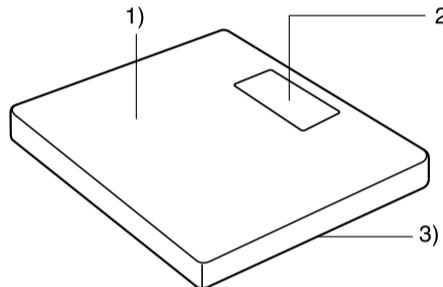
## Digital Bathroom Scale Model HD-350/355/356

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### 1. (Fig.1)
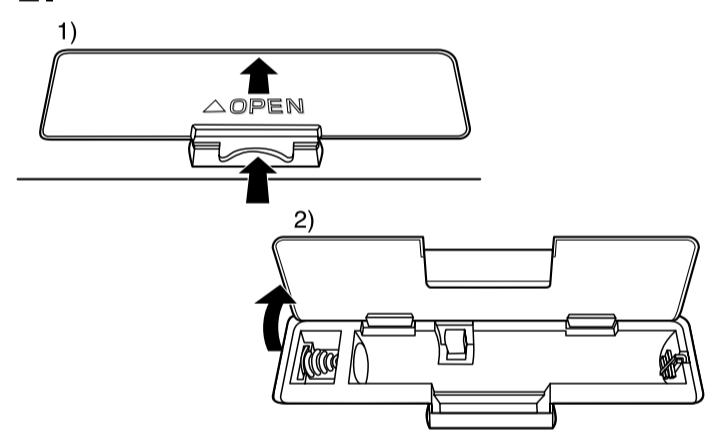
### 2. (Fig.2)

**GB**  
This device features radio interference suppression in compliance with EC Regulation 2004/108/EC.

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©2008 Tanita Corporation. All Rights Reserved.  
HD3557801(0)

## GB INSTRUCTIONS FOR USE

### 1. Specifications:

Capacity : 150 kg / 23 st 8 lb / 330 lb  
Graduation : 0.1 kg / 0.2 lb / 0.2 lb  
Battery Type : AA x 2

### 2. Parts (Fig.1)

- 1) Body
- 2) Display
- 3) Battery (Bottom)

### 3. Before Use

Select your preferred weight mode using the Weight Mode Switch.

### 4. Weighing

Press the "ON / OFF" switch.  
When "0.0" appears in the display, step onto the platform.  
LCD display "OL", if the weight is over 150 kg / 23 st 8 lb and the scale turns off after 30 seconds.

### 5. Replacing the Batteries

When the low battery sign, "Lo", appears on the LCD, please replace with new battery as indicated. (Fig.2)

### 6. Care Instructions

1. Do not attempt to open the scale or tamper with its mechanism or circuitry. These actions will void all warranty coverage.
2. Take care not to spill water or chemical liquids onto the scale. Wipe the scale with a damp cloth periodically.
3. If not in use for long term, remove the batteries to prevent damage.
4. The scale should be used on a hard even surface for accurate weighing.
5. Not recommended for use on carpeted surfaces.
6. Do not take measurements while using transmitters, such as mobile phone, which may affect readings.

## Federal Communications Commission and Canadian ICES Notice

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and Canadian ICES-003. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio or television technician for help.

### Modifications

The FCC requires the user to be notified that any changes or modifications made to this device that are not expressly approved by Tanita Corporation may void the user's authority to operate the equipment.

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