

Digital Bathroom Scale Model HD-380/381/382


- JP** 取扱説明書
- Eng** INSTRUCTIONS FOR USE
- D** BEDIENUNGSANLEITUNG
- F** NOTICE D'UTILISATION
- NL** AANWIJZINGEN VOOR GEBRUIK
- I** ISTRUZIONI PER L'USO.
- E** INSTRUCCIONES DE USO
- Br** MANUAL DE INSTRUÇÕES
- Ru** ИНСТРУКЦИИ ПО ПРИМЕНЕНИЮ
- CT** 説明書
- Ko** 사용설명서

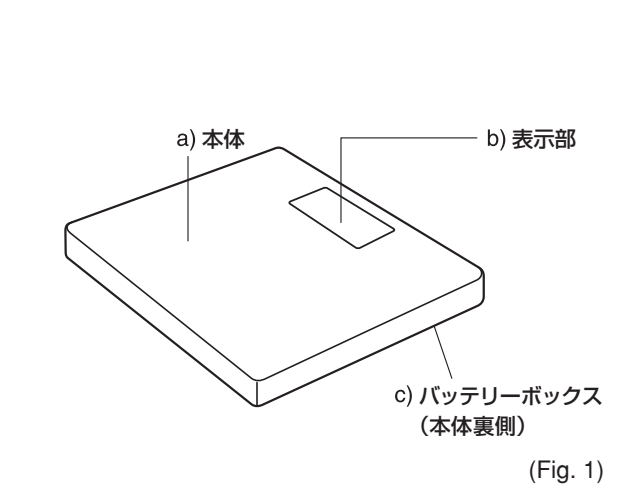(Fig. 1)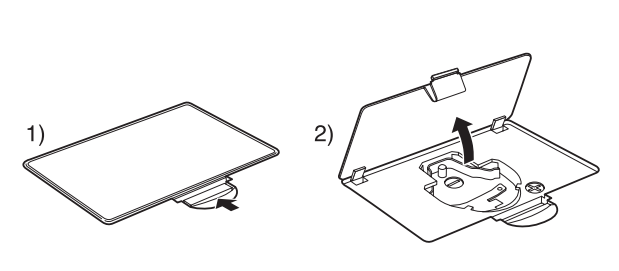

(Fig. 2) (HD-380, 381)

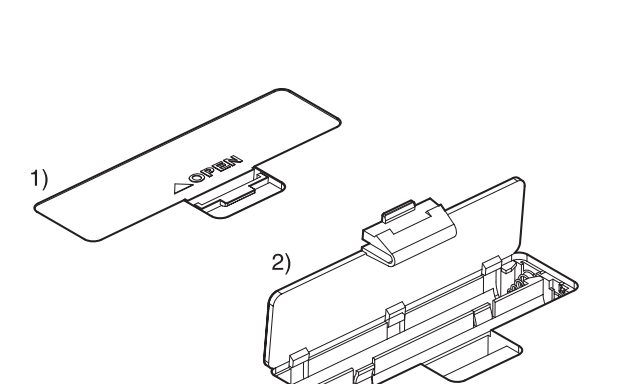(Fig. 3) (HD-382)

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- a) Body
- b) Display
- c) Battery (Bottom)

3. Before Use

Select your preferred weight mode by using the Weight Mode Switch.

4. Weighing

- Gently push down the middle of the platform with your foot and release the pressure to activate the scale.
- When "0.0" appears in the display, step onto the platform, and stand still.
- Within seconds your weight will lock and then flash.
- The scale will turn off automatically after 30 seconds.

Note: The LCD will display "OL", if the weight is over 150kg/ 330lb/ 23st 8lb, and will turn off after 30 seconds. If irregular display appears while measuring, reset the scale by removing and reinserting the batteries.

5. Replacing the Batteries

When the low battery sign (Lo) appears on the LCD, remove the current batteries and replace them with new set as indicated in Fig 2 and 3.

6. Care Instructions

- Be careful not to open the scale or tamper with its mechanism or circuitry. These actions will void all warranty coverage.
- Take care not to spill water or chemical liquids onto the scale. Wipe the scale with a damp cloth periodically.
- When not used for long periods of time, remove the batteries to prevent damage.
- Do not use this scale while operating transmitters (such as a cell phone), as this may affect your result.
- Do not step on the platform when wet.
- Do not drop any objects onto the platform.

Federal Communications Commission and Canadian ICES Notice

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and Canadian ICES-003.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

- Consult the dealer or an experienced radio or television technician for help.

Modifications

The FCC requires the user to be notified that any changes or modifications made to this device that are not expressly approved by Tanita Corporation may void the user's authority to operate the equipment.

 The device features radio interference suppression in compliance with EC regulation 2004/108/EC

 Notice: These marks can be found out on contaminant-containing Batteries:
Pb Pb = Battery contains lead,
Cd Cd = Battery contains cadmium,
Hg Hg = Battery contains mercury.

- ▲ Not allowed to mix batteries with consumer wastes!**

As consumer you are legally bound to return used or discharged batteries. You can deposit your old batteries at the public collecting points in your town, or wherever the corresponding batteries are sold and specifically marked collecting boxes have been set up. In case of scrapping the apparatus, the batteries should be removed from it and deposited at the collecting points as well.

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©2009 TANITA Corporation. All Right Reserved.